

Asepti High pH

Premium high alkaline medical washer disinfectant detergent for cleaning and compliance

ARTG 231267



Features

- Asepti High pH is the product of choice where cleaning and compliance of instruments and medical devices is the key criteria.
- Asepti High pH is a high alkali caustic-based detergent that is suitable to use on most reusable medical and laboratory instruments, including all stainless steel instruments, plastic hollow ware, anaesthetic masks and circuits. Also suitable for cleaning of dialysis units.
- Asepti High pH incorporates unique surface-active technologies to improve soil penetration and "rigid plastic" compatibility.
- Asepti High pH is suitable for use in medical batch washers, index washers and trolley washer disinfectors.
- Asepti High pH alkaline detergent complies with AS/NZS 4187:2014 and AS/NZS 4815:2014.

| Product Code | Product Description | Pack Size | Quote |
|--------------|---------------------|-----------|---|
| 5602601 | Asepti High pH, 5 L | 2 Pk |  |

Safety Information

Do not mix with any other products. For further information, safety instructions and health warnings, see product label and Safety Data Sheet.
 ALWAYS READ THE LABEL. FOLLOW DIRECTIONS OF USE.

Directions For Use

Automated applications:

- Pre-rinse all instruments with cold water.
- Recommended dilution is 3 – 8 mL per litre.

- Wash at 55°C – 80°C for 3 -10 minutes.
- In use Asepti High pH will provide a pH of between 11 – 11.5 in the wash.
- Detergent residues must be removed from instrument surfaces by successive rinses.
- Removal of residual alkalinity may be assisted by use of suitable neutralising agent such as Asepti Neutraliser.
- Asepti High pH is recommended to be used in conjunction with a lubricant or rinse aid such as Asepti Rinse Clear, Asepti Citric Rinse or Asepti Finish in the final disinfection stage of non-anaesthetic cycles.

Relevant Product

- | | |
|-------------------------------|---------------------------------|
| • Smeg GW0160 | • Smeg GW6290 |
| • Smeg GW2145 | • Smeg GW6010 |
| • Smeg GW4060 | • Smeg GW7010 |
| • Smeg GW4190 | • Smeg GW7015 |
| • Smeg GW3060 | • Belimed WD290 |
| • Smeg GW4090 | |

Applications

- Laboratory Infection Control
- Healthcare Facilities
- Central Sterile Departments
- Life Science

Resources

- safety data sheet (sds)
- Video
- Brochure