

Asepti Sonic

A non enzymatic mildly alkaline detergent and is the product of choice where economy and safety are the key criteria

ARTG R 122628




Features

- Asepti Sonic is a mildly alkaline detergent suitable for cleaning steels, soft metal substrates, plastic and elastomers and provides excellent cleaning of all medical, surgical and laboratory instruments.
- Asepti Sonic is classified as NON Dangerous and NON Hazardous.
- Asepti Sonic complies with AS/NZS 4187:2014 and AS/NZS 4815 Standards.

Safety Information

Do not mix with any other products. For further information, safety instructions and health warnings, see product label and Safety Data Sheet.

ALWAYS READ THE LABEL. FOLLOW DIRECTIONS OF USE.

| Product Code | Product Description | Quote |
|--------------|---------------------|---|
| 7100741 | Asepti Sonic, 5 L |  |

Directions

Manual cleaning and soaking

- Pre-wash all grossly soiled items with cold-ambient running water.
- Fill sinks with warm water.
- Asepti Sonic is dosed at 4-20mL per litre
- For routine soils dose at 4-10mL per litre,
- For heavy soils such as orthopaedic sets, 10-20mL per litre.
- If pumping manually, swirl to mix. Automatic dispensers are available.
- Determine solution replacement cycle and monitor.

Endoscope pre-cleaning and cleaning

- Prepare a fresh solution for each scope by diluting Asepti Sonic at 4-20 mL per litre at 30 – 50°C.
- Soak disposable cloth in bath and wipe away visible soil from the insertion tube endoscope exterior.
- Place biopsy tip into solution and aspirate detergent through suction/biopsy channel. Remove tip from bath and alternate the suctioning of air and detergent until expelled solution is visibly clean.
- Completely immerse the endoscope in the bath. Whilst immersed brush all channels and exterior surfaces and then flush and fill all channels with Asepti Sonic solution.

Ultrasonic cleaning

- Dose Asepti Wash at 2mL to 10mL per litre.
- Dispensing via an automatic wall mounted unit is recommended.
- Determine solution replacement cycle and monitor.
- Asepti Sonic is supplied with two manual pumps delivering approximately 25mls per push.
- Rinse scope, channels, removable parts with water to remove all detergent residues prior to disinfection. Discard all used solutions and materials.
- Automatic Dispensers which control product use and concentration are available. Please contact Ecolab Healthcare for further information.

Relevant Product

- [UW565 Ultrasonic Cleaner](#)
- [Bandelin Sonorex](#)
- [Bandelin Sonomic](#)
- [Bandelin Triston](#)
- [Bandelin Sonoboard](#)

Applications

- Infection Control
- Healthcare Facilities
- Residential Aged Care
- Endoscopy
- Life Science

Resources

- Video
- Brochure