


Soluscope P


Peracetic acid-based formula with proven anti-microbial activity for disinfecting endoscopes

ARTG 118778

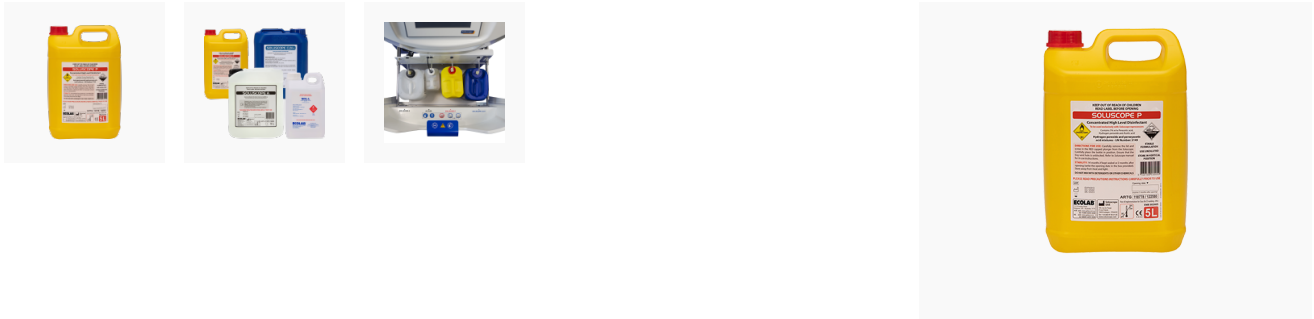

Features

- Soluscope P is a peracetic acid-based solution which has proven anti-microbial activity in 5 minutes in Soluscope Sprint, Soluscope Sprint-PT and Soluscope Series 3 PAA
- Soluscope P is compatible with all types of flexible endoscopes available on the market and complies with EN ISO 15883-4
- Soluscope P offers a specific connection as to avoid confusion with the other chemistry used on the AER such as the additive and the cleaner solution for the washer-disinfector

Product Code	Product Description	Pack Size	Quote
SL-P5	Soluscope P, Disinfectant, 5 L	2 Pk	

Safety Information

Do not mix with any other products other than the manufacturers recommendation. For further information and safety instructions see product label and Safety Data Sheet.
 ALWAYS READ THE LABEL, FOLLOW DIRECTIONS FOR USE.

Characteristics

- Clear, colourless solution
- Density: 1.10 – 1.13
- pH (pure product): 1.5 – 3.0
- Stability: 14 months (closed), 2 months (opened)

Directions For Use

- Concentrated solution for professional use exclusively in Soluscope Automated Endoscope Reprocessors: Sprint and Sprint PT
- Don the required PPE.
- Screw the plunger, equipped with the red cap, onto the bottle.
- Make sure that the vent hole is not blocked.
- Put the bottle in place.
- Store between 5°C and 25°C.

Relevant Product

- [Soluscope Sprint](#)
- [Soluscope Sprint PT](#)

Resources

- SAFETY DATA SHEET (SDS)
- Video
- Brochure